


Protect your new tattoo with Tegaderm Roll

Tegaderm is a very thin, transparent and elastic polyurethane film. The film is used to protect new tattoos against fluids, bacteria and viruses. Tegaderm allows the skin to breathe unlike plastic films or foils.

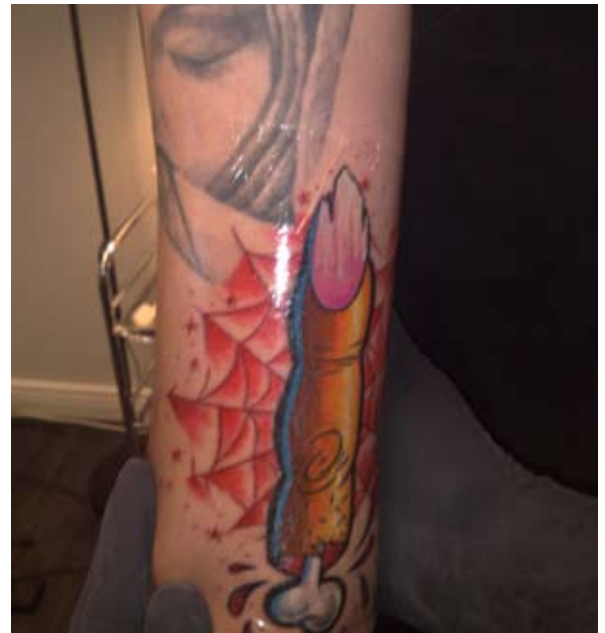

An intact dressing will protect the skin against external contamination. As Tegaderm keeps fluids out it is possible to shower with it, however avoid baths and sunbathing. The thin film is both elastic and flexible, therefore comfortable to wear.

Application



1. Make sure the skin around the tattoo is clean, dry and free from vaseline or other products that may influence adhesion.



2. Pull out preferred length from the roll and cut off



3. Remove the outer protecting film marked 1.



4. Hold the dressing by the sides and apply over the skin area.



5. Remove the side edges from the middle outwards. Pull of the side edges on the upper side of the film from middle and out.




6. Apply a little pressure using your hand over the whole film (including the edges) to ensure a tight fit Push carefully with the hand over the total film and the edges to tighten the dressing.

If the width is not enough it is possible to use another piece of film to cover a larger area.




Removing the dressing


Grab the edge of the film and stretch it off whilst using your other hand to support the skin. By stretching the dressing when taking it off reduces any potential skin damage. Grab hold of the edge of the film and stretch off the dressing and with the other hand support the skin. By stretching the dressing and expand the material easier the removal and minimize wear of the skin.

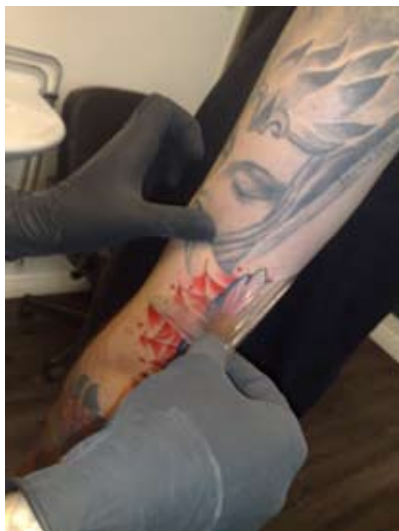

Tegaderm lets the skin breathe, unlike plastic film or foils.

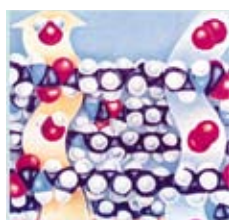

Tegaderm releases moisture from the skin yet lets Oxygen in.

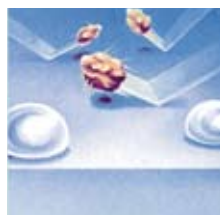

Tegaderm keep bacteria, viruses and water out.

Order from



Beställningsinformation	
3M Art nr	Storlek
16006	15 cm x 10 m

Caution:

This is not a sterile product and should not be used where sterile dressings are recommended or on infected skin.

